# Aspirator (SUCTION UNIT)

## Assembly, Installation and Operating Instructions

Model No. IRC1135

Model No. IRC1136

### IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

Read all instructions before using the Aspirator (Suction Unit).

#### DANGER

##### TO REDUCE THE RISK OF ELECTROCUTION

ALWAYS unplug this product IMMEDIATELY after using.

DO NOT use while bathing.

DO NOT place or store product where it can fall or be pulled into a tub or sink.

DO NOT reach for product that has fallen into water. UNPLUG IMMEDIATELY.

DO NOT place in or drop into water or other liquid.

#### WARNING

##### TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS

A product should NEVER be left unattended when plugged in.

Close supervision is necessary when this product is used by, on or near CHILDREN AND PHYSICALLY CHALLENGED INDIVIDUALS.

Use this product only for its intended use as described in this manual. Use this product only under physician's direction. DO NOT use attachments not recommended by the manufacturer.

NEVER operate this product if it has a damaged cord or plug. If it is not working properly, if it has been dropped or damaged, or if it has fallen into water, return the product to a service center for examination and repair.

Keep the cord away from HEATED surfaces.

NEVER use while sleepy or drowsy.

### IMPORTANT SAFEGUARDS (CONTINUED)

#### WARNING (CONTINUED)

NEVER drop or insert fingers or any other object into any openings.

NEVER block any air openings of the product or place it on a soft surface where the opening may be blocked. Keep all air openings free of lint, dirt and other foreign objects.

DO NOT use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered in a closed environment such as an oxygen tent.

This unit is oil-less. DO NOT lubricate.

Connect this product to a properly grounded outlet only. Refer to GROUNDING INSTRUCTIONS.

This unit may be thermally protected and can automatically restart when the protector resets. Always disconnect power source before servicing.

All electrical products generate heat. To avoid injury, NEVER touch the motor housing during or immediately after operation.

#### SAVE THESE INSTRUCTIONS

#### CAUTION

Risk of electric shock. Do not disassemble. Refer servicing to qualified service personnel.

#### WARNING

" Caution: Federal law restricts this device to sale by or on the order of a physician."

IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS AND INSTRUCTIONS, CONTACT INVACARE TECHNICAL SERVICES (1-800- 832-4707) BEFORE PERFORMING THIS INSTALLATION. IMPROPER USE MAY CAUSE PERSONAL INJURY AND/OR EQUIPMENT DAMAGE.

Check all parts for shipping damage. In case of damage, do not use. Contact the Carrier/Invacare for further instruction.

## REGULATORY LISTING IN COMPLIANCE WITH:

UL 1431 - Personal Hygiene and Healthcare Appliances

UL547 - Recognized Motor Protective Devices

CSA C22.2 No. 125 - Certified Personal Health Care Equipment

NOTE: This product is intended for household use.

## GROUNDING INSTRUCTIONS (FIGURE 1)

### DANGER

Improper use of the grounding plug can result in a risk of electric shock.

NOTE: This product should be grounded. In the event of an electrical short circuit, grounding reduces the risk of electrical shock by providing an escape wire for the electrical current. This product is equipped with a cord having a grounding wire with a grounding plug. The plug must be plugged into an outlet that is properly installed and grounded.

NOTE: If repair or replacement of the cord or plug is necessary do not connect the grounding wire to either flat blade terminal. The wire with insulation having an outer surface that is green with or without yellow strips is the grounding wire.

NOTE: Check with a qualified electrician or serviceman if the grounding instructions are not completely understood, or if in doubt as to whether the product is properly grounded.

NOTE: Model IRC1135 is for use on a nominal 120 V/60Hz circuit. The IRC1135 has a grounding plug that looks like the plug illustration in SKETCH "A" of FIGURE 1. If a properly grounded (3 pin) outlet is not available, a temporary adapter, which looks like the adapter illustrated in sketches "B" and "C", may be used to connect this plug to a 2-pin receptacle as shown in sketch B. This temporary arrangement should only be used with a properly grounded outlet box. Consult a qualified electrician if you are unsure of the grounding of an outlet or outlet box. The temporary adapter should be used only until a properly grounded outlet (SKETCH "A") can be installed by a qualified electrician. The green colored tab (rigid ear, lug or tie like) extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by the screw.

If it is necessary to use an extension cord, use ONLY a three (3)-wire extension cord that has a three (3)-blade grounding plug, and a three (3)-slot receptacle that will accept the plug on the product. Replace or repair a damaged cord.

Note: Model IRC1136 is for use on a nominal 230 V/50 Hz circuit. The IRC1136 model has a CEE 7/7 European plug type.

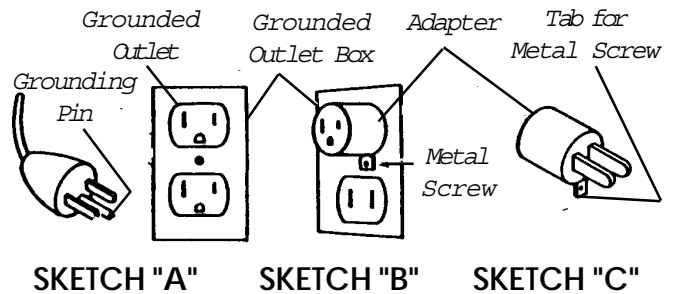
FIGURE 1 - GROUNDING INSTRUCTIONS

## ELECTRICAL

### Compressor

If the overload protector shuts off the motor frequently, you may have a low voltage situation. Low voltage can also be suspected when:

1. The motor does not get up to full power or speed.
2. Fuses or circuit breakers activate when starting compressor.
3. Lights dim or remain dim when compressor is started.
4. Other motor operated appliances fail to operate properly. Too many motor operated appliances on same circuit.

NOTE: The motor of this compressor has a thermal overload protector. If the motor should over-heat, the overload protector will shut off the motor. If this should occur, turn the ON/ OFF switch to the OFF position and allow the motor to cool (approximately 5 minutes). Restart compressor. If the compressor fails to start, wait an additional 5 minutes before attempting to restart. If this fails, check for a blown fuse on the household electric circuits. If the unit still does not restart, the motor must be replaced. Call your dealer immediately.

### Extension Cords:

NOTE: Make certain extension cord is in good condition and is NOT LESS than 18 gauge for 25 feet, 16 gauge for 50 feet and 14 gauge for 100 feet.

## OPERATION (FIGURE 2)

### WARNING

Proper operation of this product is necessary to avoid physical harm. Improper use can cause personal injury and/or tissue damage. This product should not be used unless the operator has been instructed by a physician or other qualified health care professional.

1. Familiarize yourself with the Aspirator, FIGURE 2.
2. Attach the suction tubing to the Suction Inlet.
3. Check to see that the ON/OFF Switch is in the OFF position.
4. Turn the Control Knob fully counterclockwise.
5. Plug power cord into electrical outlet.
6. Turn the ON/OFF Switch to the ON position.
7. Turn the Control Knob clockwise to adjust the Vacuum Gauge to the setting specified by your physician or qualified health care professional.

*NOTE: As the knob is turned counterclockwise the vacuum will decrease, as the knob is turned clockwise the vacuum will increase.*

### CAUTION

The Collection Jar should be emptied after every use. NEVER let the fluid level exceed the "Safe Full" line. Overflow can damage the Aspirator.

8. After each use, empty the Collection Jar. Refer to CARE AND CLEANING in this instruction sheet.

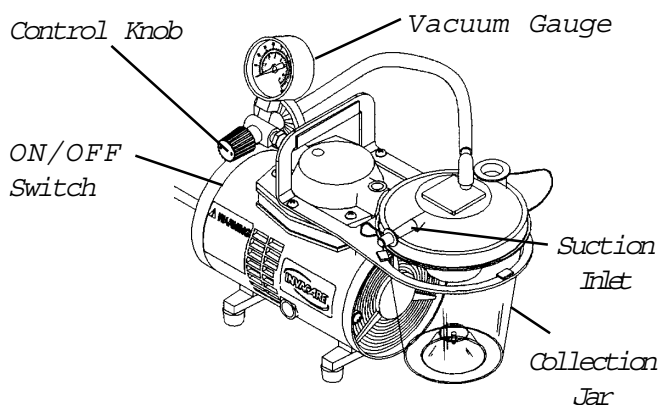
FIGURE 2 - OPERATION

## REPLACING THE HYDROPHOBIC FILTER TUBING ASSEMBLY (FIGURE 3)

*NOTE: Either of the following conditions may require replacement of the Hydrophobic Filter Tubing assembly: a) the filter is dirty and the aspirator has lost suction. b) there is liquid in the hydrophobic filter tubing assembly due to extended use or overflow.*

1. Remove the Elbow Connector end of the Hydrophobic Filter Tubing assembly from the top of the Collection Jar.
2. Unthread the Hydrophobic Filter Tubing assembly from the Vacuum Gauge and discard.

### CAUTION

Use only a hydrophobic bacteria filter. DO NOT replace with other types of bacteria filters.

3. Install new hydrophobic filter tubing assembly by reversing STEPS 1-2.

*NOTE: Hand tighten the **NEW** hydrophobic filter tubing assembly into the Vacuum Gauge. A loose filter may decrease suction.*

*NOTE: If necessary, twist the elbow connector to align with the collection jar.*

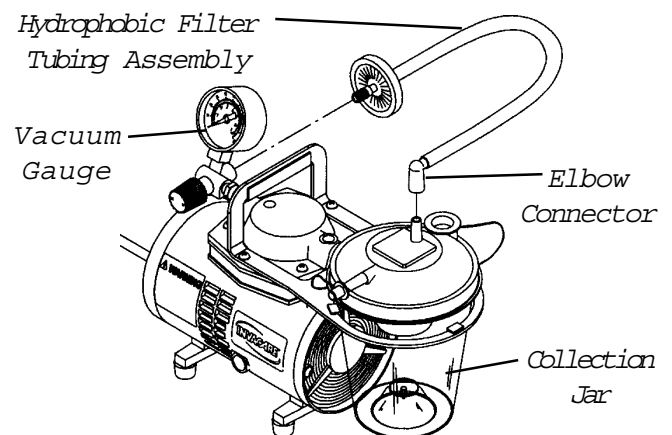
FIGURE 3 - REPLACING THE HYDROPHOBIC FILTER TUBING ASSEMBLY

---

## TROUBLESHOOTING

The suction of the aspirator has decreased or has stopped:

1. Turn the aspirator OFF.
2. Unplug power cord from electrical outlet.
3. Remove collection jar, empty and reinstall. Refer to EMPTYING THE COLLECTION JAR in this instruction sheet.
4. Refer to OPERATION, in this instruction sheet, to check suction of the Aspirator.

*NOTE: If the suction still does not return, the bacteria filter and tube assembly may need to be replaced. Refer to REPLACING THE HYDROPHOBIC FILTER TUBING ASSEMBLY in this instruction sheet. If the suction on the aspirator still has not returned after replacing the hydrophobic filter tubing assembly. Turn off the unit and call your Invacare Dealer or contact Invacare technical services at (800-832-4707).*

## CARE AND CLEANING

### Emptying the Collection Jar

#### CAUTION

The Collection Jar should be emptied after every use. NEVER let the fluid level exceed the "Safe Full" line. Overflow can damage the Aspirator.

1. Turn the aspirator OFF and allow the vacuum to return to zero (0).
2. Remove the elbow connector and tubing from the top of the collection jar.
3. Lift jar out of support bracket.
4. Remove lid by turning counterclockwise. Empty jar.

*NOTE: It is recommended that the collection jar be cleaned after every use. Refer to the CLEANING THE COLLECTION JAR in this instruction sheet.*

5. To reinstall collection jar, reverse STEPS 1-4.

## Cleaning the Collection Jar

### HAND WASH.

1. Vigorously wash separated collection jar and lid in detergent and water solution.

*NOTE: When cleaning collection jar lid, make sure the check valve is clean and freely moves to operate.*

2. Soak for 20 minutes in a solution of 2% vinegar water (acetic acid). A 2% vinegar water solution may be made by using 4 teaspoons (20cc) of white vinegar in 1 quart (1 liter) of cool water.
3. Remove jar and lid. Air dry on a piece of paper toweling.

### MACHINE WASH.

1. The collection jar is safe to put in the upper rack of a dishwasher with a water temperature maximum of 65°C or 150°F.

## REPLACEMENT PARTS

1. Hydrophobic Filter Tubing Assembly
2. Collection Jar/6 ft Suction Tubing

---

## LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the compressor of the Invacare Aspirator (Suction Unit) Compressor to be free from defects in materials and workmanship for a period of one (1) year from date of purchase. The nebulizer chamber, nebulizer air tubing, and any other accessories are excluded in this warranty. If within such warranty period any such product shall be proven to Invacare's satisfaction to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include normal wear and tear or shipping charges incurred in replacement part installation or repair of such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement.

For warranty service, please contact Invacare's Service Department during normal working hours at the Toll Free Number listed.

Upon receiving notice of an alleged defect in a product, Invacare will issue a serialized return authorization. It shall then be the responsibility of the purchaser to return, at purchaser's cost, to either Invacare's plant or service center as specified by Invacare in advance. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without prior consent. C.O.D shipments will be refused - please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT APPLY TO YOU.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2001 Invacare Corporation
Invacare is a registered trademark of Elyria, Ohio
Yes, you can. is a trademark of Invacare
Invacare Corporation [www.invacare.com](http://www.invacare.com)
Invacare Corporation.
Corporation.